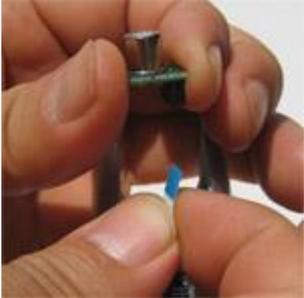
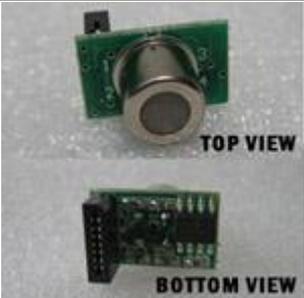
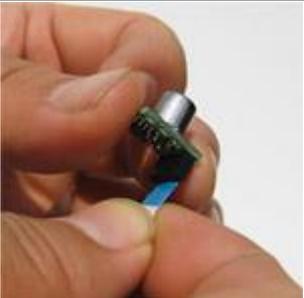


How do I install a Pre-Calibrated Alcohol Sensor Module?

For the AL6000 please take the following steps to install a new sensor module

(Click on the images for larger versions):

 <p>Prepare your AlcoMate Prestige.</p>	 <p>Use a flat, sharp tool to carefully open the top cover.</p>	 <p>Lift the breath tube to access the sensor module.</p>
 <p>Remove the breath tube. It should remove easily.</p>	 <p>Carefully raise the sensor module from the body. Do NOT detach the cable from the body.</p>	 <p>Remove the cable from the slot underneath the sensor module.</p>
 <p>Prepare a new pre-calibrated sensor module.</p>	 <p>Insert the cable into the new sensor. Make sure the blue strip is toward the sensor, as shown.</p>	 <p>Replace the sensor module into its original position.</p>
 <p>Re-attach the breath tube directly on to the sensor. Make sure the front opening is aligned.</p>	 <p>Re-attach the cover on to the top of the body. It should snap in fairly easily.</p>	<p>Ready for Testing!</p>