CERA-PET™ Blood Glucose Monitoring System User Manual

REF P13110

1 Before Use

Thank you for choosing the CERA-PET™ Blood Glucose Monitoring System. This User Guide outlines the handling precautions and correct usage procedure. Please read this user guide in full before using the CERA-PET™ Blood Glucose Monitoring System. Blood glucose monitoring should be carried out with the guidance of a veterinarian/veterinary surgeon/vet.

1-1. Intended Use

- The CERA-PET™ Blood Glucose Monitoring System is intended for use outside the body only (in vitro diagnostic use).
- The CERA-PET™ Blood Glucose Monitoring System is intended for use in monitoring blood samples taken from dogs and cats.
- Use only the CERA-PET™ Blood Glucose Test Strip with this test meter.
- The CERA-PET™ Blood Glucose Test Strips are not intended for human blood glucose testing and may give inaccurate results when used on human blood.
- For accurate testing with fresh whole blood samples (Venous, capillary blood).
- Do not let the device come into contact with hot surfaces.
- Do not try to disassemble, repair or remodel this device.
- Severe dehydration and excessive water loss may cause false, low results.
- Do not use the device if it is not working properly, or if it has suffered any damage.
- Please consult with a veterinarian before use. Any change in medication, based on the CERA-PET™ blood glucose test results, without the consent and advice of a veterinarian/veterinary surgeon, is not recommended.
- Do not use to diagnose diabetes without consulting your veterinarian.
- If your pet's blood glucose results are higher or lower than usual but your pet does not exhibit any unusual symptoms, repeat the test for accuracy. If your pet has symptoms or you continue to get results higher or lower than usual, follow the treatment advice of your pet healthcare professional.
- Conduct control solution test as directed in the user guide.

1-2. Important Test Strip Information

- Keep the test strips away from direct sunlight and heat.
- Store the test strips at a temperature between 1°C-32°C (34°F-90°F).
- Store the test strips in their original vial only. Do not transfer them to a new vial or another container.
- Clean and dry your hands before removing a strip from the vial.
- Close the vial cap tightly immediately after removing a test strip.
- Do not bend, cut or alter test strips.
- Test strips are single use only. Do not reuse.
- Drying agents in test strip vial may be harmful if inhaled or swallowed.
- Keep away from children. If this product is mishandled, it may cause serious harm or injury.

*Please refer to the insert paper in your test strip package for additional important information.

2 Table of Contents

- | | |
|--|---|
| 1. Before Use | 7. Before Testing |
| 1-1. Important Safety Information | 7-1. Coding the Test Meter |
| 1-2. Important Test Strip Information | 7-2. Testing Blood Glucose |
| 2. Table of Contents | 8-1. Getting a Drop of Blood |
| 3. Introduction | 8-2. Performing a Blood Glucose Test |
| 3-1. Intended Use | 8-3. Glucose Control Solution Test |
| 3-2. Principle of Measurement | 9. Memory |
| 4. Contents of the System | 9-1. Reviewing Results from the memory |
| 5. Understanding the CERA-PET™ Blood Glucose Monitoring System | 9-2. Clearing the Memory |
| 5-1. Appearance and Key Functions of the Test Meter | 9-3. Blood Glucose Monitoring Chart |
| 5-2. Explanation of the Test Meter Display | 10. Transferring stored Data |
| 5-3. Test Strip Description | 11-1. Maintenance |
| 5-4. Code Key Description | 11-2. Storage and Handling |
| 6. Preparation Before Use | 11-3. Error Message and Troubleshooting |
| 6-1. Changing the Battery | 12. Specifications |
| 6-2. Test Meter Set-Up | 13. Warranty |
| | 14. Definition of Symbols |

3 Introduction

The CERA-PET™ Blood Glucose Monitoring System achieves the highest standards in accuracy, reproducibility and portability. Stored test results are easily retrievable so that you may compare them to laboratory results with your veterinarian. This allows for better management of your pet diabetes and periodic blood glucose testing. Please read this user guide before using the CERA-PET™ Blood Glucose Monitoring System.

3-1. Intended Use


The CERA-PET™ Blood Glucose Monitoring System is intended for use in veterinary and medical professional setting to monitor blood glucose level in dogs and cats. It is also intended for use in the home setting to monitor blood glucose levels in dogs and cats. The CERA-PET™ Blood Glucose Monitoring System is intended for in vitro diagnostic use.

3-2. Principle of Measurement

The test is based on the measurement of an electrical current generated by the reaction of glucose with the reagent of the test strip. The test meter measures the current and displays the amount of glucose in the blood sample.


4 Contents of System

- | | |
|--|-----------------------|
| ① Test Meter | ⑦ Sterile Lancet |
| ② Lancing Device | ⑧ Carrying Case |
| ③ 3V Lithium Coin Battery | ⑨ Diabetes Diary |
| ④ User Guide | ⑩ Quick Guide |
| ⑤ Test Strips | ⑪ Code Key (Dog, Cat) |
| ⑥ Glucose Control Solution (sold separately) | |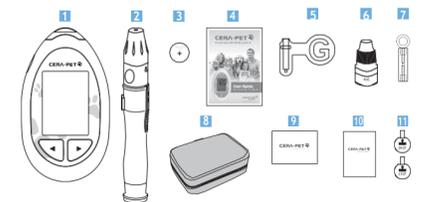


5 Understanding the CERA-PET™ Blood Glucose Monitoring System


5-1. Appearance and Key Functions of the Test Meter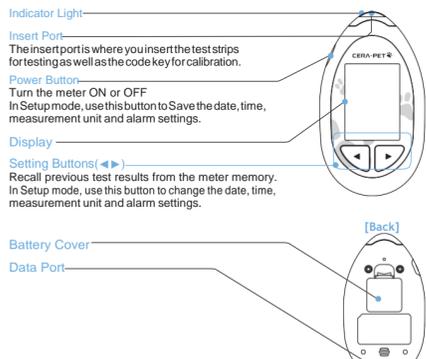

5-2. Explanation of the Test Meter Display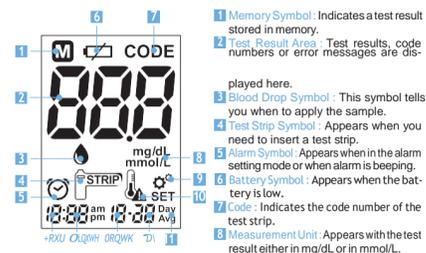


SET: Appears when in the setting mode.

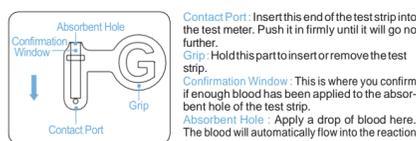Contact Port: Insert this end of the test strip into the test meter. Push it in firmly until it will go no further.
Grip: Hold this part to insert or remove the test strip.
Confirmation Window: This is where you confirm if enough blood has been applied to the absorbent hole of the test strip.
Absorbent Hole: Apply a drop of blood here. The blood will automatically flow into the reaction zone.

6 Preparation Before Use

6-1. Changing the Battery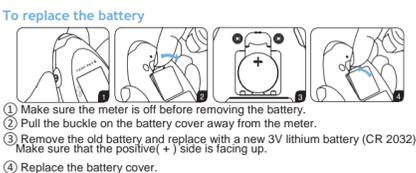

CAUTION
 Replacing the battery will not delete test meter settings unless you touch the metal wire inside the battery unit with your fingers or with another metal.
 • Exceed 1 minute when replacing the battery.
 • Load the battery incorrectly.
 • Load a used or weak battery.

WARNING
 • Keep batteries away from children's reach.
 • If a battery is swallowed, please consult a doctor immediately.


NOTE
 Recheck and reset the clock setting as necessary after battery replacement to ensure time is set correctly. To set the meter clock, see Test Meter set-up on the next page.

NOTE
 • To save battery power, the test meter automatically turns off after 2 minutes of inactivity.
 • When the low battery symbol appears (see section "11-3. Error Message and Troubleshooting") on other displayed data, it means the battery is running low but has enough power to run 20 times. It is recommended that you should replace the battery as soon as possible. The battery symbol will only appear on its own if the battery is too low to perform any more blood glucose tests. The meter will not function until the battery is replaced.
 • Discard old battery as per local guidelines.

6-2. Test Meter Set-Up

With your meter turned on or off, press the Power button for 3 seconds to enter the setup mode. The meter will also enter the set-up mode when the battery is replaced. To exit the setup mode at any stage, press the Power button for 3 seconds. The meter will save the settings and will turn off.

IMPORTANT
 Please always set the date and time before using the CERA-PET™ Blood Glucose Monitoring System for the first time or after replacing the battery. Do not perform a blood glucose test until you have set-up the test meter.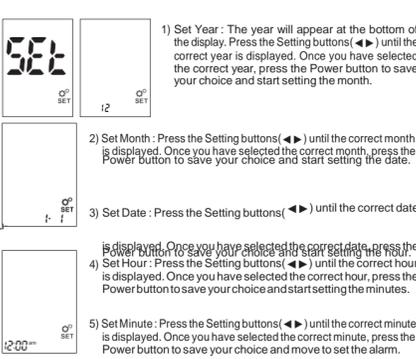

NOTE
 Pressing the Setting buttons (◀▶) continuously will pass the numbers quickly.

NOTE
 Default alarm times are 6:30, 9:30, 12:30, 18:30 and 21:30. You can choose from these or set your own time.

NOTE
 Your test meter displays 7, 14 and 30 days average which you can access from the test meter memory. These averages are calculated from results obtained during 7, 14 and 30 days from the current date and time settings. When you set a new date and time, the 7, 14 and 30 day averages may change. After 2 minutes of inactivity while the test meter is in the setting mode, it will shut down automatically.

NOTE
 7) Set Indicator: Press the Setting buttons (◀▶) to select to turn On or Off the Indicator light. With the indicator On, the indicator light located on the insert port will be blinking and vice versa. After selecting the Indicator On or Off on the display, press the Power button to save your choice and move to set the measurement unit.

IMPORTANT
 Please consult with your veterinarian or veterinary surgeon about which measurement unit is most suitable for you.

NOTE
 Your test meter displays 7, 14 and 30 days average which you can access from the test meter memory. These averages are calculated from results obtained during 7, 14 and 30 days from the current date and time settings. When you set a new date and time, the 7, 14 and 30 day averages may change. After 2 minutes of inactivity while the test meter is in the setting mode, it will shut down automatically.

7 Before Testing



Each batch of the CERA-PET™ Blood Glucose Test Strip comes with two code keys; one for dogs and one for cats. The code keys are encoded with the algorithms for the blood glucose testing for a specific pet species. In order to ensure an accurate result, the code number on the display must also match the code number found on the Test Strip Vial for your pet species.

7-1. Coding the Test Meter

1) With the test meter display facing you, insert the code key completely into the code key port with the code number also facing you.
 2) The code number will appear on the LCD display.
 3) Make sure the code numbers and pet species on the display match the code key and the test strip vial for dog or cat. If all of the numbers match, you may begin testing.

NOTE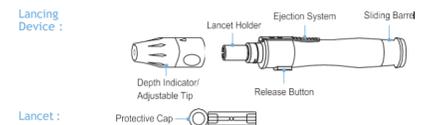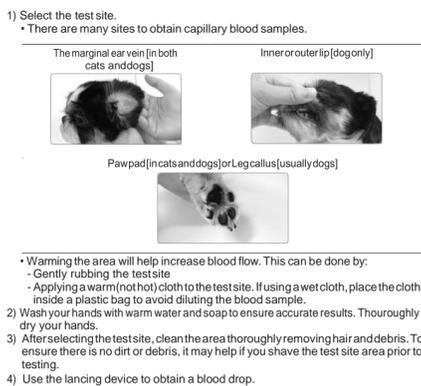


CAUTION
 1) Unscrew the cap of the lancing device and insert a lancet into the lancet holder and push down firmly until it is fully seated.
 2) Twist the protective cap until it is separated from the lancet.

NOTE
 3) Replace the lancing device cap. Twist the adjustable tip to set the depth level. The number lines up with the arrow: 1-2 for soft or thin skin, 3 for average skin, 4-5 for thick or calloused skin. Generally you will need a higher setting for a dog than a cat.
 4) Pull the sliding barrel until it clicks.

NOTE
 3) Press the tip of the device firmly against the desired puncture site and press the release button. You will hear a click, indicating that the puncture is complete.
 4) Gently squeeze the punctured area to obtain a blood sample. Once the test has been completed, gently apply pressure to the test site with sterile gauze or cotton wool to help stop the blood flow.

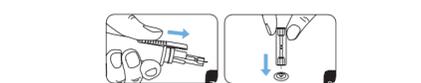NOTE
 1) Unscrew the cap of the lancing device and push the ejection system upwards to remove the lancet.
 2) Place the protective cap on a hard surface and push the exposed tip into the protective cap.

8-2. Performing a Blood Glucose Test

1) Remove a test strip from the vial and immediately close the cap. Using your right hand, hold the glucose strip by the "G" handle and insert the test strip fully into the insert port of the test meter, with both the "G" handle and the test meter display facing you. The "M" symbol will blink on the display.
 2) The code on the LCD display must match the code number on the test strip vial.

NOTE
 If you do not apply a blood sample to the test strip within 2 minutes, the test meter will automatically shut down. You must remove and reinsert the test strip to restart the test procedure.

3) Obtain a blood sample. Please refer to section of "8-1. Getting a drop of blood". You should wipe away the first drop of blood with a tissue and test using the 2nd drop.

4) Apply blood to the absorbent hole of the test strip until the confirmation window is full of blood. The test meter will begin to count down from 5 seconds and a result will be displayed. This reading is automatically stored in the test meter.

CAUTION
 The blood sample should completely fill the confirmation window in order for the countdown to begin. If you find that the confirmation window is not completely filled when the test meter is counting, DO NOT try to add more blood to the test strip. Discard the test strip and retest with a new one.

8-3. Glucose Control Solution Test

CERA-PET™ Blood Glucose Control Solution contains glucose that reacts to the test strips. By comparing the result from a control solution test with the expected range printed on the test strip vial label, you can check that the test meter and the test strips are working together as a system and that you are performing the test correctly. It is very important that you do this simple check routinely to make sure you get accurate results.

Perform a Glucose Control Solution Test if
 • You would like to test your system without using a blood sample.
 • You suspect the test meter or test strip is not working properly.
 • You are experiencing:
 • The test strip vial was not closed properly.
 • You are using a new vial of test strips.
 • You dropped the test meter

NOTE
 The results obtained from Glucose Control Solution test do not indicate your pet's glucose level.

IMPORTANT

Use the CERA-PET™ Blood Glucose Test Meter only with the CERA-PET™ Blood Glucose Test Strips. Glucose Control Solution is sold separately. Do not use the control solution after expiration date as printed on the vial. Use the control solution within 4 months after opening. It is recommended to record the opening date on the Glucose Control Solution vial as a reminder for when you need to dispose of the opened control solution.


NOTE
 Always close the cap on the Glucose Control Solution immediately after use.

1) Using your right hand, hold the glucose strip by the "G" handle and insert the test strip fully into the insert port of the test meter, with both the "G" handle and the test meter display facing you. The blood symbol will start blinking.
 2) Shake the Glucose Control Solution vial thoroughly. Remove the cap and squeeze the vial. Discard the first drop and wipe off the dispenser tip to prevent contamination.
 3) Squeeze the vial to get another drop of glucose control solution and apply directly to the absorbent hole of the test strip. The drop will be automatically drawn into the test strip. Make sure the confirmation window is completely filled.
 4) The test strip will count down from 5 seconds and then display the Glucose Control Solution test result. Compare this result with the control solution range printed on the test strip vial label.

WARNING
 If the test result is outside the range printed on the test strip vial, repeat the test. If you continue to have out-of-range results, it means that the system may not be working properly. Do not use the product and please contact your local distributor for assistance.

8 Memory


9-1. Reviewing Results from the Memory
 The CERA-PET™ Blood Glucose Test Meter can store up to 1000 test records. Each record includes the test result, time and date. The meter will also calculate the average values of records from the last 7, 14 and 30 days.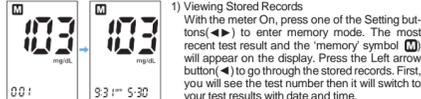

NOTE

NOTE
 If there are already 1,000 records in the memory, the oldest record will be erased to make room for a new one. Your most recent result will display first.

9-2. Clearing the Memory

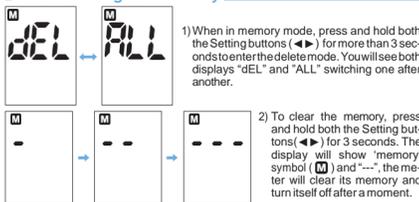3) If you entered the delete mode but want to exit without deleting the stored data, press the Power button. This will turn the meter off without deleting any data.

WARNING
 Extreme caution should be used when clearing the memory. This is not a reversible operation.

9-3. Blood Glucose Monitoring Chart

Normal Blood Glucose Level [Source] Home Monitoring of Blood Glucose Concentration in the Management of Diabetes Mellitus Home, Compendium Small Animal/Exotics Vol.23, No.6 June 2001:p544-575
 72-140mg/dL or 4.0-7.8mmol/L (Dog, Cat)

10 Transferring Stored Data

The CERA-PET™ Blood Glucose Test Meter allows the stored meter data to be transferred to your computer. Using the CERA-PET™ blood glucose management software and USB data transfer cable (purchased separately) you can transfer and show analytical interpretations.

1) Make sure the glucose meter is turned off. Data cannot transfer when the device is still on.
 2) After you initiate the software for blood glucose management on the computer, you connect the transfer cable from the computer to the glucose meter.
 3) If the connection is properly carried out, the glucose meter will display "PC" on the display screen.

NOTE
 If you disable the cable during the transfer of data, the data may not transfer properly. Please keep the transfer cable intact until the transfer is completed.

4) After the data transfer is completed, you can remove the transfer cable. If you need more details about the blood glucose monitoring software of CERA-PET™ BGMS,

please contact your local distributor.

11 Caring for your Test Meter and Test Strip

To avoid contaminating your test meter and test strips, please wash your hands and test site with soap and water. Make sure you dry your hands and the test site thoroughly.

11-1. Maintenance

Your test meter does not require special maintenance. There is no special cleaning required because no blood or control solution comes in contact with the test meter. Avoid getting dirt, dust, blood, or water inside the test meter through the Insert port. A damp cloth and mild detergent can be used to wipe the outside of the test meter. Your test meter is a precision instrument. Please handle it with care.

11-2. Storage and Handling

- Test Meter storage and handling
 - Store test meter at -10°C - 60°C (14°F - 140°F).
 - Do not leave the test meter in very hot or cold places.
 - Do not leave it near a heat source or in your vehicle during adverse weather conditions.
 - Always store or carry the test meter in its original storage case.
 - Avoid dropping and strong impact.
 - Avoid direct sunlight and humidity.
- Test Strip storage and handling
 - Store test strips at 1°C - 32°C (34°F - 90°F), below 85% relative humidity.
 - Store your test strips in their original vial only.
 - Do not transfer to other containers.
 - Store old test strips in a cool and dry place.

- Keep away from direct sunlight and heat.
- After taking test strips from the vial, close the vial cap immediately.
- Make sure your hands are clean and dry before touching the test strips.
- Use each test strip immediately after removing it from the vial.
- Write the disposal date on the test strip vial as a reminder to dispose of the opened vial after 4 months.
- Do not bend, cut, or alter a test strips in any way.
- Keep the test strips away from children since the cap and the test strips may be a choking hazard, immediately see a doctor for help.
- Follow local governing regulations and recycling plans for disposal or recycling of device components.

11-3. Error Message and Troubleshooting

Message	Possible Cause	What You Should Do
HI	Test result level is more than 900 mg/dL (50.0 mmol/L)	Repeat the test with a new test strip. If this message shows again, please contact your veterinary surgeon immediately.
Lo	Test result level is less than 10 mg/dL (0.6 mmol/L)	Repeat the test with a new test strip. If this message shows again, please contact your veterinary surgeon immediately.
BATT	Battery power is low.	Prepare to replace the battery.
LOW	The battery power is too low for testing.	Replace the battery immediately.
TEMP	The operating temperature of the system is not within the recommended range 4°C - 40°C (39°F - 104°F).	Allow the test meter and test strips to warm up or cool down slowly (30 minutes) until the error message goes away.
TEST	Test strip is used.	Repeat test with a new test strip.
Err-1	Failure to operate during a test.	Repeat test with a new test strip.
Err-3	Test strip was removed during a test.	Repeat test with a new test strip.
Err-4	Damaged code key or the code key was removed during coding of the meter.	Re-code the Test Meter. If the problem persists please contact your local distributor.
Err-7	The code is not set.	Insert the code key that accompanied the box of the test strips.
Err-8		

12 Specifications

- Product Name : CERA-PET™ Blood Glucose Monitoring System
- Model : G 300V
- Sample Type : Fresh whole blood from capillary and venous in dogs and cats
- Sample Size : 0.5µl
- Hematocrit Range : 10% - 70%
- Test Time : 5 seconds
- Dimensions : 53.6 x 94 x 13.9 (mm)
- Weight : 35g
- Power source : One CR2032 3V lithium coin battery

- Altitudes up to 4,000m (13,200ft)
- Measurement Range : 10-900 mg/dL (0.6-50 mmol/L)
- Storage/Transport Condition : -10°C-60°C (14°F-140°F)

(Blood Glucose Test Meter) below 85% relative humidity, 700 to 1,060 hPa)

13 Warranty

The manufacturer, GREEN CROSS MEDIS Corp., guarantees the CERA-PET™ Blood Glucose Monitoring System to be free of defects in material and workmanship for a period of 3 years.

This guarantee is valid from the date of purchase. The guarantee extends only to the original purchaser and is not transferable.

14 Definition of Symbols

Symbol	Reference
[VD]	In vitro diagnostic medical device
[X]	Do not reuse
[I]	Consult instructions for use
[T]	Temperature limitation
[U]	Use by
[M]	Date of manufacture
[LOT]	Batch code
[SN]	Serial Number
[P]	Power Button
[D]	Do not dispose of the instrument with your normal waste
[W]	Caution, consult accompanying documents
[DC]	Direct current

My pet's Information

- Name :
- Species : □ Dog □ Cat
- Breed :
- Gender : □ Female □ Male
- Age :
- Diabetic : □ Yes □ No
- Pet Owner's Name :
- Phone :
- Veterinary Clinic's Name :

- Clinic's Contact :
- Average Blood Glucose Level:
- Insulin shot : □ Yes □ No

- Always use the correct pet species code key for your pet to ensure your test meter gives you the most accurate result.
- If the code numbers displayed on the screen do not match the code key and the test strip vial, try calibrating again or contact your local distributor.
- Do not misplace your code keys.

Testing Blood Glucose

NOTE

- Please wash and dry your hands prior to starting.
- Use the Lancing Device to obtain a blood sample from your dog or cat.
- Consult a veterinarian/veterinary surgeon for a recommended sampling method and advice.

Temperature Symbol : Appears when the temperature is abnormally high or low.
Day Average : Indicates whether the displayed reading refers to the 7, 14 or 30 day test average.

5-3. Test Strip Descriptio

Your test strip measures the concentration of glucose in whole blood. Blood is applied to the absorbent hole of the test strip and is automatically drawn into the reaction zone.

When you press the Right arrow button (▶) for viewing data, it will display the average value of test results performed during 7 days. Pressing again the Right arrow button (▶), it shows the average value of tests during 14 and then 30 days. When you press the Left arrow button (◀), it will display the latest result.

* If you press the Power button when viewing memory, the glucose meter will close down.

- Display : LCD
- Memory : 1000 test results with date and time
- Automatic Turn Off : Shuts down after 2 minutes of inactivity
- Measurement Units : Either mg/dL or mmol/L
- Operating Condition : 4 °C–40 °C (39 °F–104 °F), below 85% relative humidity

 GREEN CROSS MEDIS Corp.
16, Jeongja 1-gil, Seongseo-eup, Seobuk-gu, Cheonan-si,
Chungcheong nam-do 31045, Korea

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