

DOWNLOAD THE APP!



PetTest
Pharma Supply, Inc.
✓ Installed



Available on the
App Store



GET IT ON
Google Play

DIGITAL CO



PetTest

DIGITAL COMPANION

SET UP

Enter your PetTest meter Serial Number

You will find your PetTest serial number printed on the back of your meter.

DIGITAL CO



Welcome to ADVOCATE PetTest

Verify your device SN




.....



Enter

SET UP

**Enter email
& password
then click
create a
new account**

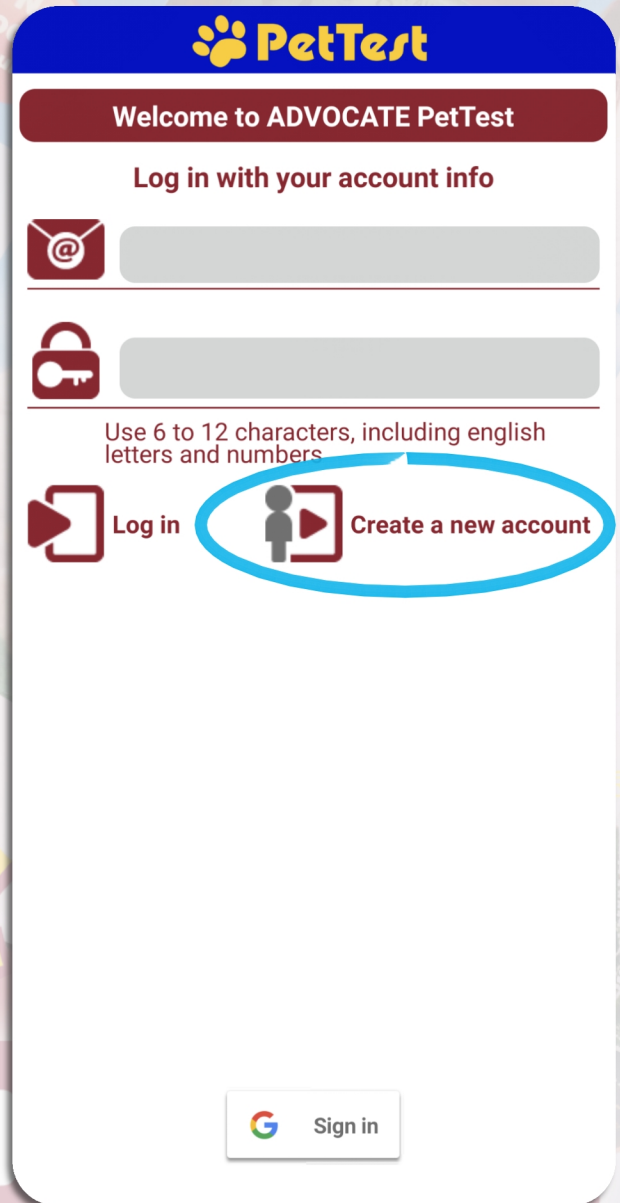


The image shows a mobile application interface for PetTest. At the top is a blue header with the PetTest logo. Below it is a dark red banner with the text "Welcome to ADVOCATE PetTest". The main content area is white and contains the following elements:

- A heading "Log in with your account info" in dark red.
- An email input field with an @ icon on the left.
- A password input field with a lock icon on the left.
- Text below the password field: "Use 6 to 12 characters, including english letters and numbers".
- A "Log in" button with a right-pointing arrow icon.
- A "Create a new account" button with a person icon and a right-pointing arrow icon. This button is circled in blue.
- A "Sign in" button at the bottom right, featuring the Google logo.

SET UP

**Verification
email will be
sent, click on
link to verify**

DIGITAL CO

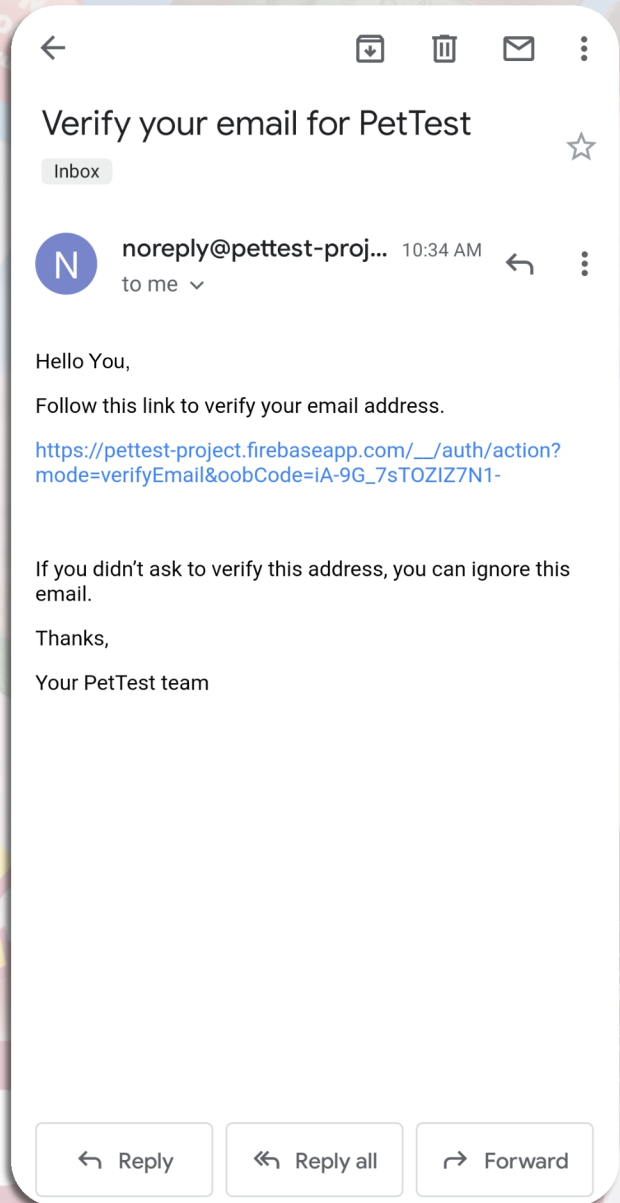

SET UP

**Enter your
email & password
then click on
LOG IN**

**Most devices will remember
and automatically save your
information for faster log in.**

DIGITAL CO



Welcome to ADVOCATE PetTest

Log in with your account info



Use 6 to 12 characters, including english letters and numbers



Log in




Create a new account



Sign in

SET UP


**Click on MENU
then click on
PROFILE to enter
your information**

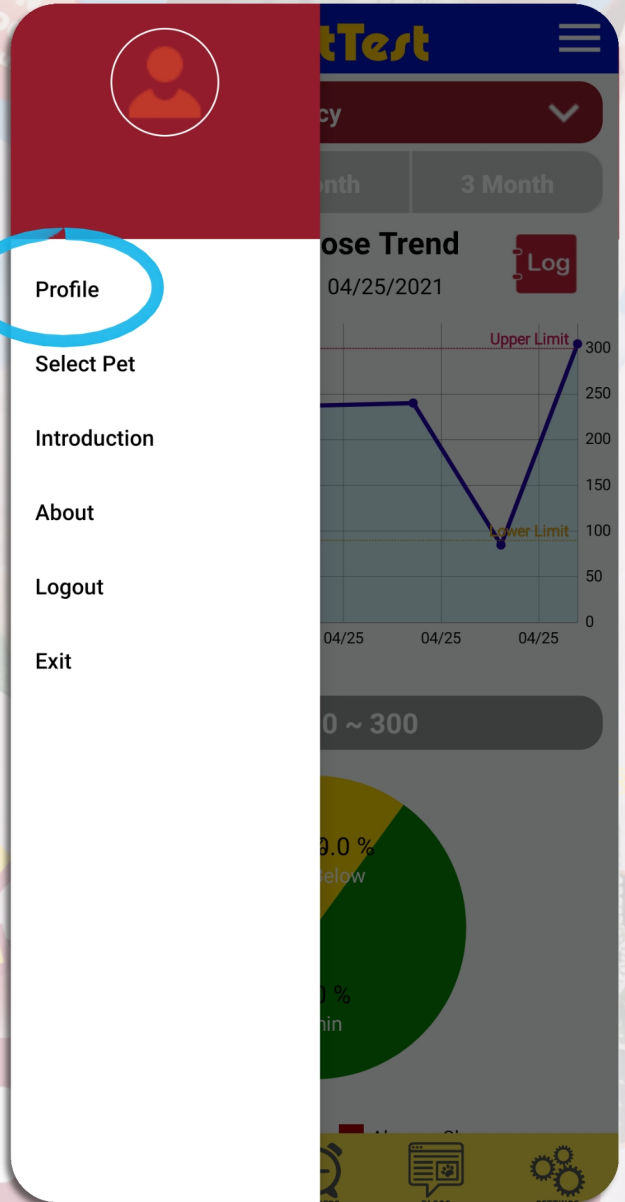

DIGITAL CO

SET UP

**Click on
SELECT PET
to enter pet's
information**

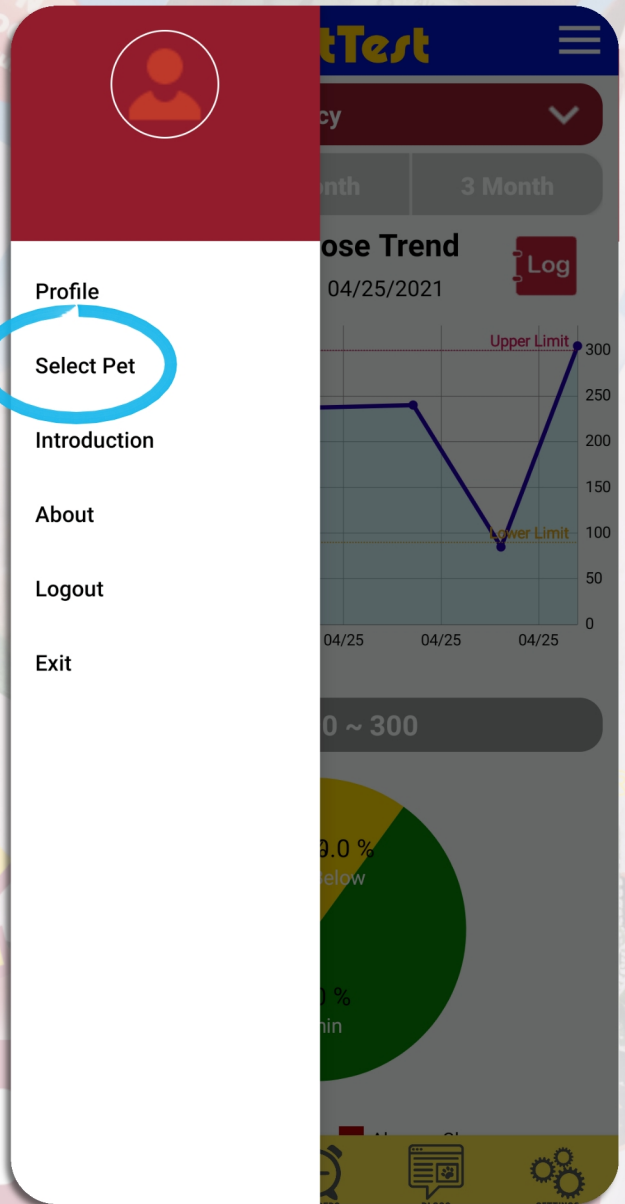

DIGITAL CO

SET UP

Add your
favorite pic
to your
pet's profile



*Name

[Click here to add pic](#)

*Species

Breed

Weight

lb

kg

Diabetes Type

NA

Type 1

Type 2

Gender

Date of Birth

*Glucose Reading Limit:

Upper Limit

mg/dL (mmol/L)

Lower Limit

mg/dL (mmol/L)

Create


Cancel

DIGITAL CO

USING YOUR APP


Select your pet for data & information

DIGITAL CO

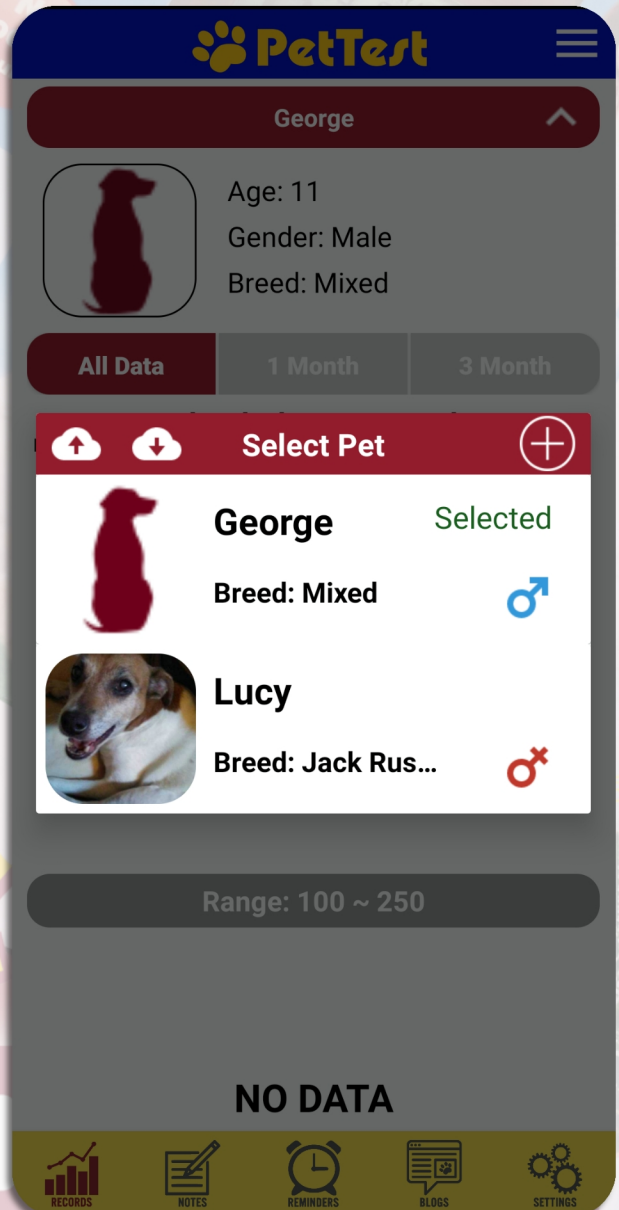

USING YOUR APP

Swipe left to edit or delete

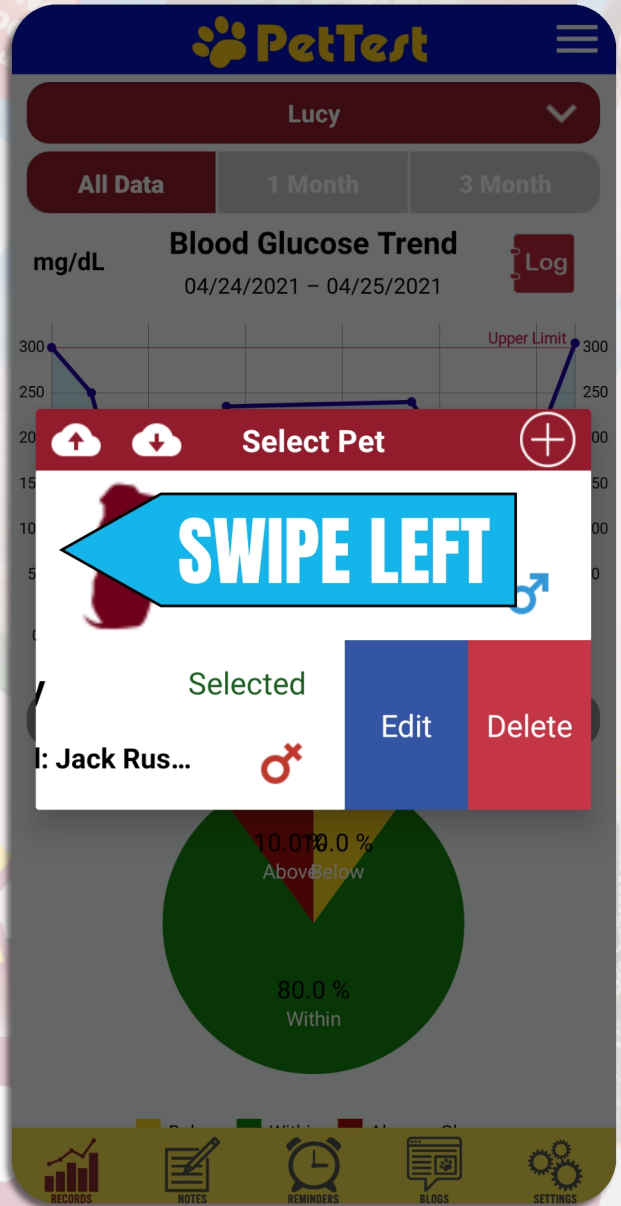

DIGITAL CO

USING YOUR APP

Add your data to your log

DIGITAL CO

Back 

From 04/24/2021 To 04/25/2021


Date	Time	Value
04/24/2021	07:00	300
04/24/2021	09:43	250
04/24/2021	11:00	175
04/24/2021	13:00	125
04/24/2021	15:00	150
04/24/2021	17:00	180
04/24/2021	19:00	235
04/25/2021	13:46	85
04/25/2021	19:00	305

[Click here to add data](#)



USING YOUR APP

Data collected in RECORDS!

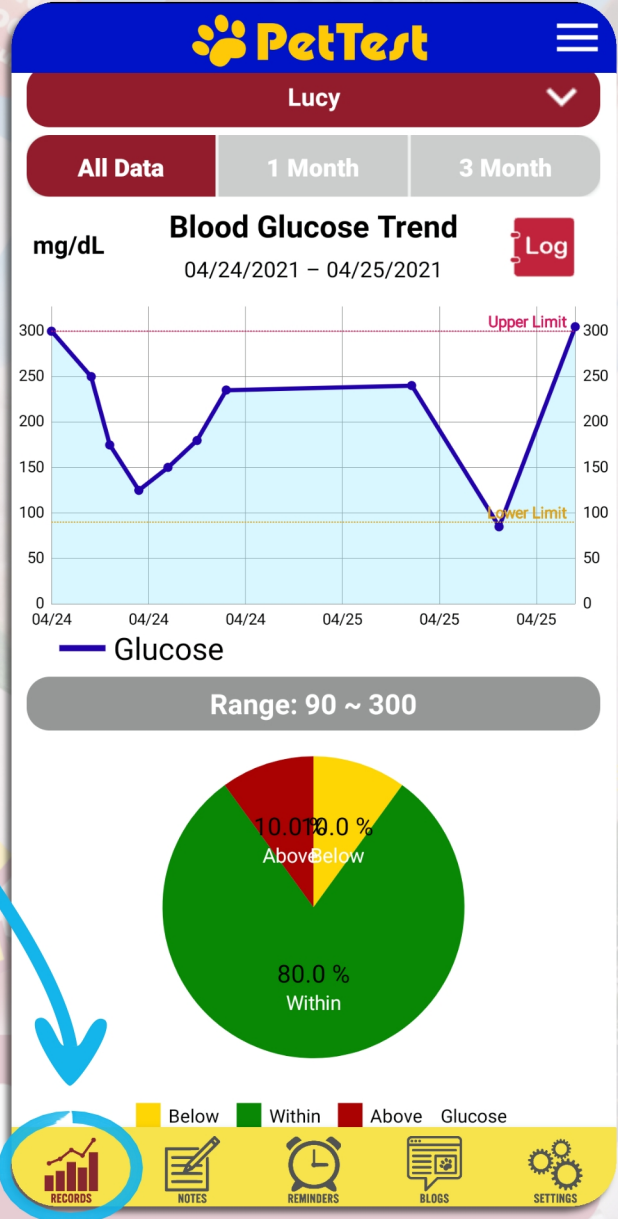

USING YOUR APP

**Add NOTES
in this
section**


The screenshot displays the PetTest app interface. At the top, the PetTest logo is visible. Below it is a search bar with the text "What are you looking for" and a search icon. A "TODAY" button is on the left, and a "+" button is on the right. The main content is a calendar for April 2021. The calendar shows days of the week (Sum, Mon, Tue, Wed, Thu, Fri, Sat) and dates. The date 24 is highlighted with a red box. At the bottom, there is a navigation bar with five icons: RECORDS (bar chart), NOTES (notepad with pencil, circled in blue), REMINDERS (alarm clock), BLOGS (speech bubble with paw print), and SETTINGS (gears).

Sum	Mon	Tue	Wed	Thu	Fri	Sat
28	29	30	31	1	2	3
4	5	6	7	8	9	10
11	12	13	14	15	16	17
18	19	20	21	22	23	24
25	26	27	28	29	30	1
2	3	4	5	6	7	8

DIGITAL CO

USING YOUR APP


**Add REMINDERS
with alarms
here!**

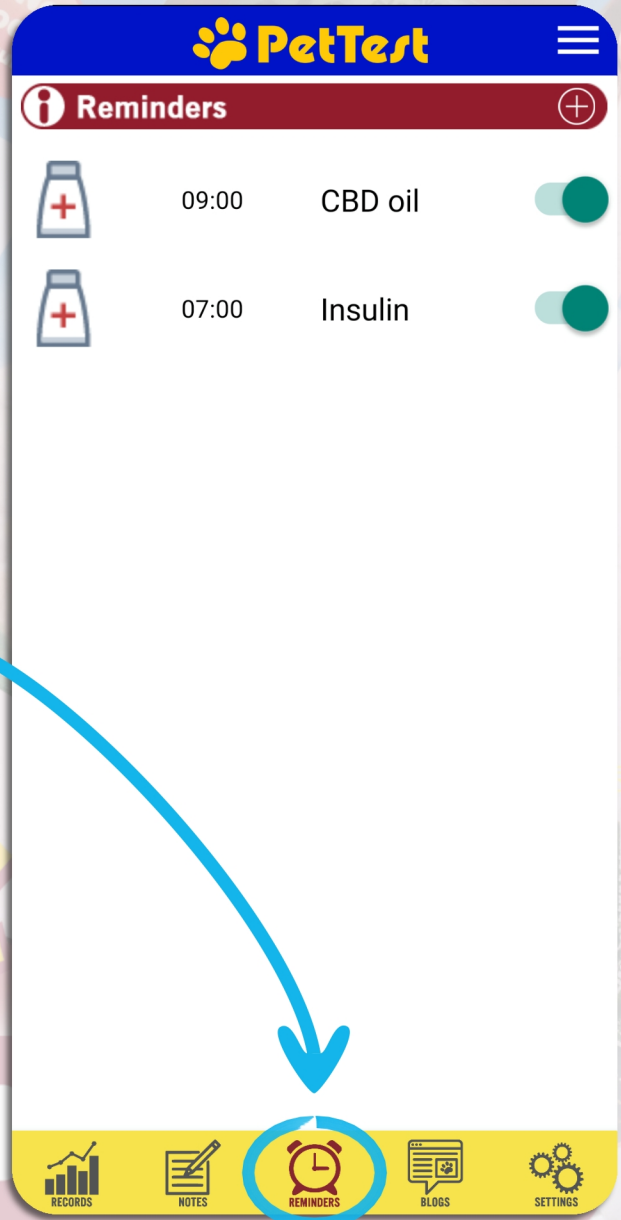

DIGITAL CO

USING YOUR APP

Change your
SETTINGS here

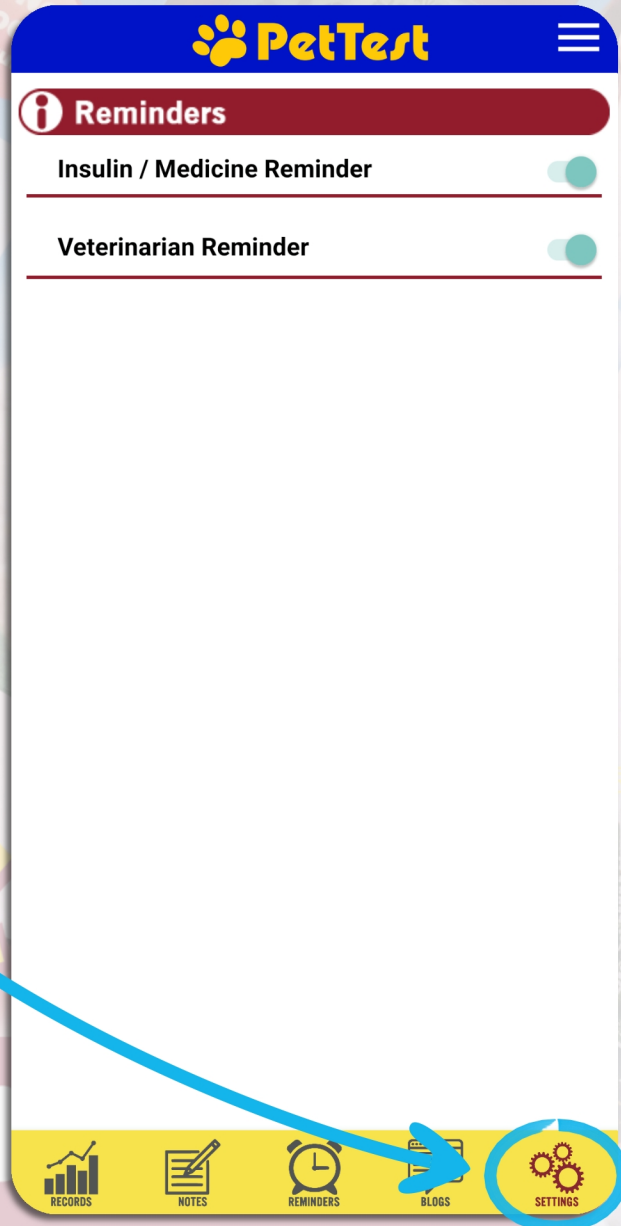

DIGITAL CO

USING YOUR APP

Click on **BLOGS** to take you to PetTest Blogs



RECORDS



NOTES




REMINDERS



BLOGS



SETTINGS

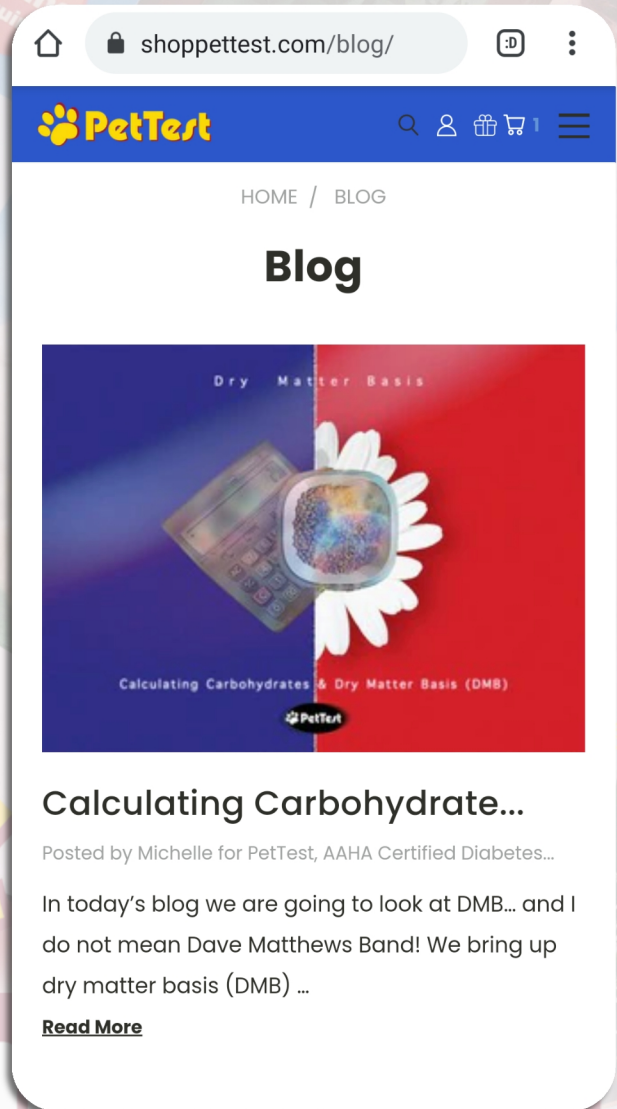

Calculating Carbohydrate...

Posted by Michelle for PetTest, AAHA Certified Diabetes...

In today's blog we are going to look at DMB... and I do not mean Dave Matthews Band! We bring up dry matter basis (DMB) ...

[Read More](#)

DIGITAL COMMUNICATION