

clearblue DIGITAL OVULATION TEST

Before applying urine to the Test Stick it must be inserted into the Test Holder. Read the instructions carefully before doing the Test.

How Clearblue Digital Ovulation Test works with you

Clearblue Digital Ovulation Test works by detecting the increase (surge) in the hormone called Luteinising Hormone (LH) in your urine. The LH surge occurs approximately 24-36 hours prior to the release of an egg from your ovaries – a process known as ovulation. You are at your most fertile on the day your LH surge is detected, and the day after.

Your 2 most fertile days begin from the time that Clearblue Digital Ovulation Test detects your LH surge. Make love at any time in the next 48 hours to maximise your chances of becoming pregnant.


When to start testing

The day that the LH surge occurs varies from woman to woman and from one cycle to another. To have the best chance of detecting your LH surge with the number of Test Sticks provided in this pack, you need to know your usual cycle length. To work out your cycle length, count the day your period starts (first day of full menstrual flow) as Day 1, and continue counting through to the day before your next period starts. The number of days is your cycle length.

If you do not know your usual cycle length, refer to Question 1 overleaf.

Use the table below to work out the day you should start testing. You can test any time of day, but it should be at about the same time each day (many women find it most convenient first thing in the morning). It is important that you have not urinated for at least 4 hours before testing, and you should avoid excessive fluid intake before testing.

Length of your cycle (days)	21 or less	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41 or more
Count the first day of your period as Day 1, start testing on the day shown under your cycle length.	5	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	See Question 2 overleaf.

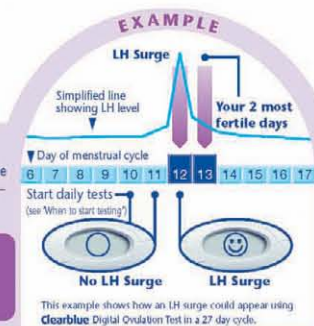

1 When you are ready to test...

- Remove the Test Stick from the foil wrapper.
- Before applying urine to the Test Stick it must be inserted into the Test Holder. Follow the steps below.




2 Remove the Cap

- Find the Pink Arrow on the Test Stick.



3 Assemble the Test

- Find the Pink Arrow on the Test Holder.
- Line up both Pink Arrows.
- Insert the Test Stick into the Test Holder until it clicks into place and the Test Ready symbol appears.
- Do not use until the Test Ready symbol appears.
- When the Test Ready symbol appears do the Test straight away.

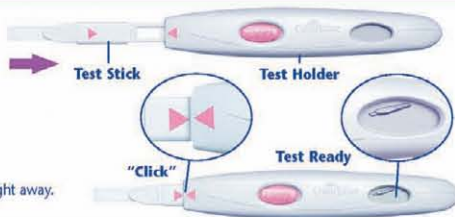


- If the Test Ready symbol does not appear OR if the symbol has disappeared eject the Test Stick and re-insert it carefully.
- If the symbol still does not appear refer to Question 19 overleaf.

4 Do the Test

With the Test Ready symbol showing on the Display...

- Either test directly in your urine stream.
- Place just the Absorbent Sampler, pointing downwards in your urine stream for 5-7 seconds.
- Take care not to get the Test Holder wet.

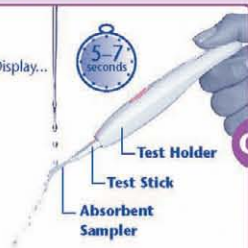

- OR test a sample of your urine collected in a clean, dry container.

- Dip just the Absorbent Sampler in your collected urine sample for 15 seconds.



5 Wait

- Keep the Absorbent Sampler pointing downwards or lie the Test flat.
- The Test Ready symbol will start flashing after 20-40 seconds to show the Test is working.
- DO NOT EJECT THE TEST STICK.




- Replace the Cap then wipe off any excess urine.
- If the Test Ready symbol does not start flashing, refer to Question 6 overleaf.


6 See your result

- Within 3 minutes the Display will show your result.



- If your result is  you have not detected your LH surge. Test again tomorrow at the same time using a new Test Stick.



- If your result is  you have detected your LH surge and have reached your most fertile time. Make love at anytime during the next 48 hours to maximise your chances of becoming pregnant. There is no need to carry on testing during this cycle.

- Once you have read your result you **MUST** press the Eject Button and remove the Test Stick.
- Your result will remain on the Display for 8 minutes.
- The result of your Test can only be read on the Test Holder Display.
- If your Test Stick is still inserted and the Display is now blank, eject the Test Stick and your result will re-appear for 2 minutes.
- You cannot determine your result from any lines that you see on the Test Stick.
- Do not re-insert a used Test Stick.

Error symbols

Error A

- You have ejected the Test Stick too soon. Re-insert the Test Stick into the Test Holder immediately. Refer to Question 16 overleaf.



Error B

- Refer to Question 17 overleaf.



Error C

- Refer to Question 18 overleaf.



FURTHER INFORMATION

Disposing of your Digital Test

- Once your result has been displayed, dispose of the Test Stick with your normal household waste. To do this, press the Eject Button on the Test Holder. The Test Stick will automatically be released. When you eject the Test Stick you may notice one or two blue lines on the Display on the Test Holder. The Test Stick is not reusable.
- Once you have used all the Test Sticks in the pack, remove the batteries from the Test Holder by separating the upper and lower halves starting at the end nearest the Display. Remove the two round silver batteries from under the central metal cover, using a pen. Dispose of the batteries according to the appropriate recycling scheme. **Caution:** Do not disassemble, recharge, or dispose of the batteries in fire. Do not swallow. Keep away from children. Dispose of the rest of the Test Holder with your normal household waste. Do not dispose of electrical equipment in fire.

Using the Test Holder Again

- Whilst doing a test the Test Holder may have been splashed with some urine. Ensure that you wipe it clean. A further test is not possible whilst your result or any error symbol are on the Display. These will be displayed for 8 minutes (or 2 minutes if your Display was blank and you have just ejected the Test Stick). It is important that you have not urinated for at least 4 hours before doing a test. If you have collected your urine you can use the Test Holder again as soon as the Display is blank. When you are ready to use the Test Holder again, repeat the instructions from 1 using a new Test Stick.

QUESTIONS & ANSWERS

1. I do not know my usual cycle length.

How do I know when to start testing?

If you don't know your usual cycle length you are advised to wait for at least one menstrual cycle, and note the length of this cycle, before using Clearblue Digital Ovulation Test. Once you have determined your cycle length you can use the instructions overleaf to work out when to start testing.

If you wish to start using Clearblue Digital Ovulation Test before knowing your cycle length we recommend you start testing on Day 10. (However, without working out your cycle length, it is possible you may miss your LH surge because you have started testing too late or you may need to start a new pack to continue testing. Please refer to Question 10.)

2. My cycle length is outside of the range shown in the table. How will I know when to start testing?

If your cycle is 21 days or less, start testing on day 5. If your cycle is longer than 40 days, start testing 17 days before you expect your next period.

3. What time of day should I carry out the Test?

You may test any time of day, but you should test at approximately the same time each day. You should not have urinated for at least 4 hours before testing.

4. Do I need to use all the tests?

No. You can stop testing when you detect your LH surge, and save the remaining Test Sticks and Test Holder to use next cycle if necessary.

5. Can I use the Test Holder with any other Test Sticks?

You can only use Clearblue Digital Ovulation Test Sticks with this Test Holder. If you have Test Sticks left from a previous Clearblue Digital Ovulation Test Pack you can use them with the Test Holder in this pack.

6. I have done a test, but the Test Ready symbol has not started flashing. What does this mean?

The Test has not been done correctly. An error symbol will appear on the Display within 10 minutes of testing. You should re-test with a new Test Stick. For information on re-testing please refer to Q17 and 'Using the Test Holder again'.

7. What if the Test Holder gets wet?

If the Test Holder gets wet you should wipe it clean. Some splashing will not damage the Test Holder, but if it becomes very wet it may be damaged. If so, the Display will either remain blank or an error symbol will be displayed.

8. I have done a test, but no result has appeared on the Display. What does this mean?

Your result should appear on the Display within 3 minutes of doing a test. If no result appears an error symbol will appear on the Display within 10 minutes. Please refer to Question 17.

9. How accurate is Clearblue Digital Ovulation Test?

The Clearblue Digital Ovulation Test has been shown in extensive laboratory trials to be over 99% accurate in detecting the LH surge prior to ovulation. The sensitivity of the Clearblue Digital Ovulation Test is 40mIU/ml.

10. I've done all the tests as instructed, but I've not yet detected my surge. What should I do?

The number of Test Sticks in a Clearblue Digital Ovulation Test pack is sufficient for most women with regular cycles to detect their LH surge. If your cycle length varies by more than 3 days then it is possible that you may need to do more tests to detect your LH surge. You will need to start a new pack and use the new Test Holder and Test Sticks to continue testing. Refill packs containing only Test Sticks are not available. Some women do not ovulate each cycle and therefore will not see an LH surge in such cycles.

11. Can any medication or medical conditions affect the result?

- Always read the manufacturers instructions for any medication that you are taking before conducting a test.
- Certain medical conditions and medications can adversely affect the performance of the Test: for example if you are actually pregnant, have recently been pregnant, have reached the menopause or have polycystic ovarian syndrome you may get a misleading result. This may also be true if you are taking fertility drugs containing Luteinising Hormone or human Chorionic Gonadotrophin. Please check with your doctor.
- Clomiphene citrate does not affect the tests, but may affect the length of your cycle and, therefore, when you should be testing. You may need to start a new pack and use the new Test Holder and Test Sticks to continue testing.

12. I've recently stopped using hormonal contraception (e.g. the pill). Will this affect the results?

No, it will not affect your results. However your natural hormone pattern is disrupted by hormonal contraception and if you have recently stopped using it

your cycles can be irregular and may take some time to stabilise. You may wish to wait until you have had two natural menstrual cycles, and note the length of these cycles, before using Clearblue Digital Ovulation Test.

13. Since it tells me when I am fertile, can I use Clearblue Digital Ovulation Test as a contraceptive?

No. Clearblue Digital Ovulation Test detects the LH surge, which occurs only 24-36 hours before ovulation. As sperm can survive for several days, you might still become pregnant if you made love before you detected your LH surge.

14. I think I have become pregnant, what should I do?

You can find out if you are pregnant from the day your period is due by using a Clearblue Digital Pregnancy Test. A Clearblue Digital Pregnancy Test is over 99% accurate, easy to use and gives your result in words. If your Clearblue Digital Pregnancy Test shows you a 'Pregnant' result you should see your doctor who can advise you on what steps you should take next. The earlier you see your doctor, the better it is for your own health.

15. I've used Clearblue Digital Ovulation Test for several months and haven't become pregnant. Can I be sure of getting pregnant?

It can take many months to become pregnant. There are many reasons why you may not become pregnant even if you've been able to make love at your most fertile time. If after several months you've had no success, you should consult your doctor.

Error Messages

16. A flashing symbol (Error A) has appeared on the Display. What does this mean?



The Test Stick has been ejected too soon. If it is not re-inserted quickly enough then a further error symbol will appear on the Display. Please refer to Question 17.

17. This symbol (Error B) has appeared on the Display. What does it mean?



An error has occurred during testing. It may be that

- Urine has been applied to the Absorbent Sampler before the Test Stick was inserted into the Test Holder.
- The Absorbent Sampler was not kept pointing downwards, or the Test was not laid flat, after the urine was applied.
- Too much or too little urine was applied.
- The cartridge was ejected too soon. This symbol will be displayed for 80 minutes. You should test again using a new Test Stick taking care to follow the instructions. Please refer to 'Using the Test Holder again'.

18. This symbol (Error C) has appeared on the Display. What does this mean?



Please contact the Clearblue Careline for further information. If you have unused Test Sticks do not use them with this Test Holder.

19. The Display has remained blank. What does this mean?



If you have not applied urine to the Test Stick eject it and start again from step 3 overleaf. Otherwise, please contact the Clearblue Careline.

For further information about Clearblue, please call the Clearblue Careline. A 24 hour, 7 days a week recorded information service is available for your use. In addition, the Clearblue team is available to answer your questions and give you confidential advice from 9am – 3pm Mon-Fri, excluding Bank Holidays.

☎ (08705) 673514 UK ☎ 1890 882 885 IRL

Calls charged at local rate and may vary by network operator. Please check with your operator for exact charges.

Please be ready to quote the lot number **LO1** on the pack. Calls may be recorded or monitored for quality control.

For next steps, information and general advice relating to issues of pregnancy, visit our website at:

www.Clearblue.info

As prenatal care is very important for a baby's health, we recommend that you consult your doctor before you try to conceive.

*Home ovulation test.
For in vitro diagnostic use only.
Store at room temperature.
Keep out of the reach of children.
Do not use if the foil wrapper
containing the Test Stick is damaged.
Do not use a Test Stick that has
passed its expiry date.
These instructions are for
Unipath model number 505121.*



EMC - Guidance and Manufacturer's Declaration

The Clearblue Digital Ovulation Test is intended for use in the electromagnetic environment specified below. The customer or the user of the Clearblue Digital Ovulation Test should assure that it is used in such an environment. This product is tested to EN60601-1-2:2002.

Electromagnetic Emissions			Electromagnetic Immunity		
Test	Compliance	Electromagnetic Environment – Guidance	Test	Compliance Level	Electromagnetic Environment – Guidance
RF emissions CISPR 11	Group 1	The Clearblue Digital Ovulation Test uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.	Electrostatic discharge IEC61000-4-2	±6kV contact ±8kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
RF emissions CISPR 11	Class B	The Clearblue Digital Ovulation Test is suitable for use in all establishments, including domestic establishments.	Radiated RF IEC61000-4-3 a)	3V/m (from 80MHz to 2.5 GHz)	d = 1.2·P from 80 MHz to 800 MHz d = 2.3·P from 800 MHz to 2.5 GHz where P is the maximum power rating of the transmitter in watts and d is the recommended separation distance from the transmitter in metres.

NOTE a). These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.