TRIGLYCERIDES

Test Strips to use with PRIMA® meter
REF-PL303 - 25 strips, 1 data-chip
REF-PL304 - 10 strips, 1 data-chip

PL303_4_ifu_1.0/04

Intended use
Triglycerides Test strips are used with the PRIMA® meter for quantitative measurement of triglycerides in whole blood obtained from fingertip. The strips are for testing outside the body (in vitro diagnostic use); intended for in-home (self-testing) and point-of-care setting. Before using these test strips please read this package insert and the PRIMA® User Manual carefully.

Test Principle
Each test strip has been chemically pre-treated with enzymes. On each test strip there is a white application area containing reaction chemicals. When finger-lanced blood is applied to this area, a chemical reaction takes place and the color of the strip changes. The deeper the color the higher triglycerides level in the blood sample. PRIMA® meter measures reflectance of light caused by this color change and interprets it into the user's triglycerides value. Your blood triglycerides level is then displayed on the meter's LCD window. Triglycerides data chip (yellow) is provided with each container of triglycerides test strips and must be properly inserted into analyzer before any triglycerides test can be run. The chip contains test name, calibration curve, and lot number along with the expiration date. The test measures triglycerides from 0.56mmol/L (50 mg/dL) to 5.6 mmol/L (500mg/dL).

Strip Storage and Handling

- Do not remove the test strips from the strip container until immediately before performing a test.
- Check the expiry date of the test strips printed on the container. Do not use out-of-date strips.
- After opening a new strip container, the strips are stable for 3 months - try using the strips within three months after you first opened the container.
- This will ensure the most accurate results.
- Tightly re-cap the container immediately after removing a test strip. Exposure to light and moisture may discolor the test strip which may lead to false and inaccurate results.
- Use test strip as soon as you have removed it from the container.
- Do not cut or alter strips in any way.
- Avoid exposing the strips to extreme temperatures. Store the strip container at 5° - 30°C. Do not freeze.
- Store the strips only in the original container. Do not combine or mix with other strips.
- Do not remove or discard the desiccant bag from the container.
- Keep the strip container away from children.
- Discard test strip after measurement is completed. Do not re-use strips.
- Do not ingest.

How to do a test

Please refer to your PRIMA® meter's manual for step-by-step instructions how to operate a digital meter, use a lancing device and perform a measurement.

1) extract the green data-chip 2) insert the data-chip 3) open the vial 4) take the strip 5) immediately close the vial 6) select the gender ("F" for female, "M" for male) 7) select if before ("A") or after meal ("P") 8) wash hands 9) insert the strip in to the meter with the yellow side outside 10) check the display if there is the TGC and the number corresponds to the one printed on the vial 11) get the drop of blood with the lance 12) apply the blood on the strip 13) wait for the result.

Precautions

- Abstain from alcohol for 2 days.
- Abstain from taking vitamin supplements and particularly Vitamin C for at least 24 hours before the test.
- Consume only water before the collection of blood. Do not drink coffee or tea when you fast.

Tips for getting a good drop of blood

- The PRIMA® meter and triglycerides test strips require a large hanging drop of blood. If you are new to self-testing, or have had trouble getting blood in the past follow these steps:
 - Wash hands in warm water, rinse and dry completely. If you use a hand cream, keep in mind that almost all creams contain fat and can affect the triglycerides reading. Therefore, if you do use a hand cream, wash your hands very thoroughly and rinse carefully with clean water to obtain the most accurate results.
 - Warning your hands is always helpful with increasing blood flow to fingertips.
 - If you use an alcohol wipe to clean your finger make sure the finger is completely dry before landing because traces of alcohol would give false results.
 - Let your arm hang down at your side briefly to allow blood flow to fingertips.
 - Grasp finger near area to be pricked and squeeze gently for three seconds.
 - Lance side of the fingertip and squeeze gently until you get a drop of blood.
 - Always wipe away the first drop of blood with a cotton ball or gauze. Apply the second drop to the test strip.

What does your results mean

Triglycerides are a common form of fat that we digest. Elevated levels of triglycerides are a risk factor for heart disease, heart attack, stroke, fatty liver disease, and inflammation of pancreas (pancreatitis). Elevated levels of triglycerides are also associated with diseases like diabetes, kidney disease (nephrotic syndrome), and medications (for example, diuretics, steroids, birth control pills, and beta blockers). The U.S. National Cholesterol Education Program (NCEP) and the Canadian Cardiovascular Society (CCS) strongly recommend to do repeated measurements of triglycerides in conjunction with total cholesterol to find out your baseline (beginning) lipids value. At least 2 or 3 tests should be performed over a period of 2 or 3 weeks. The average of these values will give your baseline cholesterol and triglycerides values. The PRIMA® meter has an option to automatically average all the tests stored in the meter's memory (see your manual to set up this option). You may want to see your family doctor for more information on your baseline values. Never adjust or change your medications and diet based on your triglycerides and/or cholesterol value obtained by the meter – always consult with your doctor first.

The NCEP and CCS advise that adults have their blood lipids checked at least once every five years, starting at age 20. However, if you are over 40 you should get your triglycerides tested at least once a year. Diabetics must have their triglycerides measured every 3 months.

Guide to understand your triglycerides test results

Normal Triglycerides
If your result is below than 1.7 mmol/L (150 mg/dL) your triglycerides level is considered "normal" and at low risk. Record your result in the log book and discuss with your doctor how often you should re-test your triglycerides. Your level may change over time, especially if you drastically change your diet and exercise routine.

Suspect Borderline-High Triglycerides

If your test result is between 1.7 and 2.1 mmol/L (150 - 184 mg/dL) your triglycerides level is suspect and borderline-high, although some experts think these are acceptable values. It means you are at increased risk for developing a heart disease compared to those with lower levels. It doesn't mean you necessarily need intensive medical attention but you should consult with your doctor.

Compromised High Triglycerides

If your test result is between 2.1 mmol/L (184 mg/dL) and 5.6 mmol/L (500 mg/dL) you are in the "compromised and high risk" category for developing heart disease. One test is not enough to confirm high triglycerides. At least 2 or 3 tests should be performed over the period of several weeks. The average of these values will determine your true blood triglycerides level. You should talk to your doctor for a complete medical check-up.

Blood triglycerides level is affected by diet habits and by your body's metabolic state as well as your genetic makeup. A high triglycerides level in conjunction with high total cholesterol is one of the risk factors of heart disease. However, other risk factors which can increase the likelihood of heart disease are smoking, high blood pressure, un-balanced diet high in saturated fat, obesity, insufficient exercise, as well as alcohol abuse and diabetes. If you have any questions about your lipids level and/or risk factors, consult with your doctor.

Unusual Results

If you keep monitoring your triglycerides level with the PRIMA® meter regularly or occasionally get unusually high or low results, there may be a problem. First, check and review your test procedure, integrity of test strips or functionality of your meter (see your manual for a function check with control solutions).

- The following are the most common causes of unusually high or low results:
 - Test strips were kept in an opened container.
 - Expired test strip was used.
 - Proper maintenance and handling procedure for the digital meter have not been followed.
 - The first drop of blood was applied instead of the second. The first drop usually contains some extracellular tissue fluid that dilutes your blood drop and causes false results.
 - Contamination of the blood sample with cosmetics or hand lotions (most contain glycerol).

Limitations of the test

- Triglycerides test strips give reliable and accurate results under the following limited conditions:
 - Measurement range 0.56 – 5.6mmol/L (50 – 500mg/dL).
 - Only fresh capillary blood should be used for testing.
 - Recent acute illness, pregnancy, hormonal medications including birth control pills may change your blood triglycerides values. If you have any concerns consult with your doctor.
 - This test system has not been validated for neonates.
 - The following medications in high concentrations can alter total cholesterol levels: Vitamin C, dopamine and methyl dopa.
 - Excessive squeezing and milking of the pricked site may cause erroneous results.
 - This test should not be used by people who are on the blood thinning medication (anticoagulants) as well as by patients with bleeding disorders (hemophiliacs etc).
 - The test should be performed at room temperature (10° - 35°C).
 - The test should be performed at less than 80% humidity.

Additional Supplies

To order Triglycerides Test Strips in the 10 or 25 strips containers (REF-PL303; PL304) as well as Triglycerides Control Solution (REF-PL309) please visit our web site: www.primahometests.com.

Limited Warranty

Manufacturer of this product warrants that the test strips will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the test strips container.

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In vitro diagnostic medical device

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