PROCEDURE

- 1) Wash your hands with soap and rinse with clear warm water.
- 2) Tear the protective pouch (from the notch) and only get out the cassette and the pipette. Dispose of the small desiccant bag.
- 3) Hold the lancet and carefully rotate 360° the protective cap without pulling it. - FIG. A
- 4) Extract and dispose the released cap. - FIG. B
- 5) Press the lancet, on the side from where the cap was extracted, against the fingertip (it is advisable the ring finger side). - FIG. C The tip retracts automatically and safely after use.
- 6) Keeping the hand down massage the end that was pricked to obtain a blood drop. - FIG. D
- 7) **Without pressing the pipette bulb**, put it in contact with the drop. The blood migrates into the pipette through the capillarity to the line indicated on the pipette. - FIG. E You may massage again your finger to obtain more blood if the line is not reached. As far as possible, avoid of air bubbles.
- 8) Put the blood collected with the pipette into the sample well of the cassette, by pressing on the pipette bulb. - FIG. F
- 9) Wait for the blood to be totally dispensed in the well. **Unscrew the blue cap** of the plastic dropper (leave the white cap tightly screwed) and add **5 drops** of diluent into the sample well of the cassette. - FIG. G
- 10) Read the result after **10 minutes**.

RESULT INTERPRETATION

READ THE RESULT AFTER 10 MINUTES.
DO NOT READ THE RESULT AFTER 15 MINUTES.

The intensity of the line colour does not have any importance for the interpretation of the test result.

POSITIVE RESULT




Two coloured lines appear in the window under the marks T (Test) and C (Control). The intensity of the line T may be clearer than the intensity of the line C. This result means that the IgE antibodies level in blood is higher than the norm. You should consult a doctor as you may suffer from an allergy or from an allergic predisposition.

NEGATIVE RESULT

Only one coloured line appears under the mark C (Control). This result means that the IgE antibodies level in blood is normal and that there is no allergy sensitivity.

NON VALID RESULT

No line appears or a coloured line appears under the mark T (Test) without any line under the mark C. In this case, it is not possible to interpret the test, which must be considered as non valid. It is recommended to repeat the test with a new **ALLERGY-IgE TEST** and a sample of fresh blood.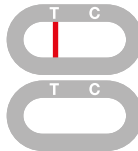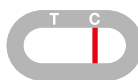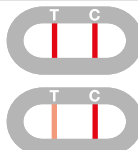

F.A.Q. - QUESTIONS & ANSWERS

HOW DOES THE ALLERGY-IgE TEST WORKS? IgE antibodies are produced by the body in reaction to the presence of one or several allergens likely to cause more or less serious complications in subjects exposed to allergenic substances. The **ALLERGY-IgE TEST** specifically identifies IgE antibodies and indicates if the level is higher than the average.

WHEN SHOULD THE TEST BE USED? The **ALLERGY-IgE TEST** should be performed in the case of symptoms that can be associated with the allergy, such as rashes and redness on the skin, lacrimation or repetitive sneezing in presence of dust or in contact with animals (dog, cat, etc.), or appearance of oedema after ingesting particular food. The test can be performed at any time, but not in subjects younger than 6 years.

CAN THE RESULT BE INCORRECT? The results are accurate as far as the instructions are carefully respected. Nevertheless, the result can be incorrect if Allergy Test gets wet before test performing or if the quantity of blood dispensed in the sample well is not sufficient, or if the number of diluent drops are less than 4 or more than 5. The plastic pipette provided in the box allows making sure the collected blood volume is correct.

HOW TO INTERPRET THE TEST IF THE COLOUR AND THE INTENSITY OF THE LINES ARE DIFFERENT? The colour and intensity of the lines have no importance for result interpretation. The lines should only be homogeneous and clearly visible. The test should be considered as positive whatever the colour intensity of the test line is.

WHAT IS THE LINE THAT APPEARS UNDER THE MARK C (CONTROL) FOR? When this line appears, it only means


that the test was performing well.

IF I READ THE RESULT AFTER 15 MINUTES, WILL THE RESULT BE RELIABLE? No. The result should be read within 10 minutes after adding the diluent. The result is reliable up to 15 minutes.

WHAT DO I HAVE TO DO IF THE RESULT IS POSITIVE? If the result is positive, it means that the IgE (Immunoglobulin E) antibodies level in blood is higher than the norm and that you should consult a doctor showing him the test result. Then, the doctor will decide whether additional analysis should be performed or will recommend you consult an allergist.

WHAT DO I HAVE TO DO IF THE RESULT IS NEGATIVE? If the result is negative, it means that the IgE (Immunoglobulin E) antibodies level in blood is normal. In principle, there is no risk of allergy sensitivity. However if the symptoms persist, it is recommended to consult your doctor.

WHAT IS THE ACCURACY OF ALLERGY-IgE TEST? The **ALLERGY-IgE TEST** is very accurate and is used for more than 10 years by professionals in the field (hospitals, laboratories...). Evaluation reports show that an overall agreement of at least 96% with reference methods.

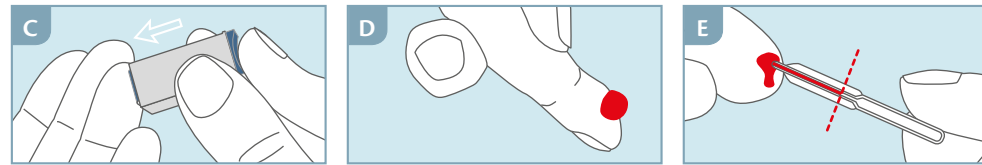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