**PRECAUTIONS**

- 1) Carefully read the instructions before performing the test. The test is only interpretable if the instructions are carefully respected.
- 2) Keep out of the reach of children.
- 3) Do not use after the expiry date printed on the label and on the protective pouch or if the pouch is damaged.
- 4) Follow strictly the indicated time, blood and diluent quantities.
- 5) Store between +4 °C and +30 °C. Do not freeze.
- 6) Do not re-use the cassette and lancet.
- 7) External use only. DO NOT SWALLOW.
- 8) This test is exclusively intended to in vitro diagnostic self-testing use.
- 9) Not recommended for use by individuals taking medicines that thin the blood (anticoagulants) or by the individuals with disorders such as hemophilia.
- 10) After use, dispose of all components in accordance with local regulations, seek advice from your pharmacist.

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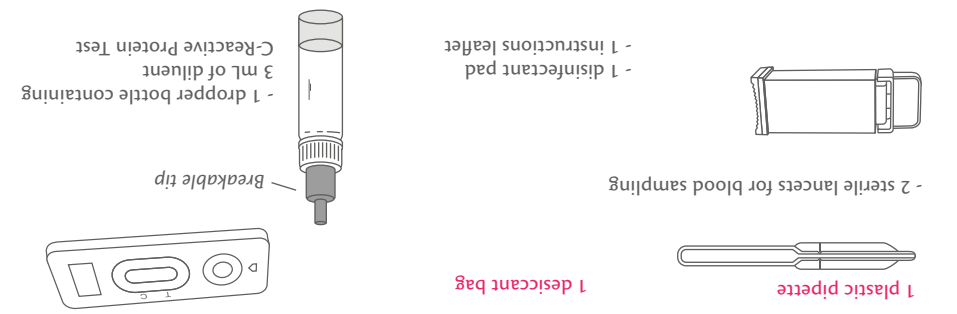
**CONTENT**

THE BOX CONTAINS ALL THE NECESSARY TO PERFORM 1 C-REACTIVE PROTEIN TEST:

- 1 sealed aluminium pouch containing:
  - 1 plastic pipette
  - 1 desiccant bag
- 2 sterile lancets for blood sampling
- 1 disinfectant pad
- 1 instructions leaflet

\* Not provided necessary material: absorbent cotton.  
 \* Only open the protective pouch when you are ready to use the test.  
 \* The desiccant bag should not be used.

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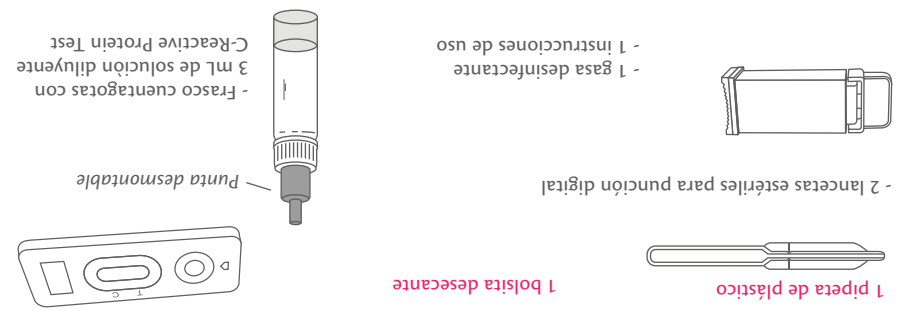
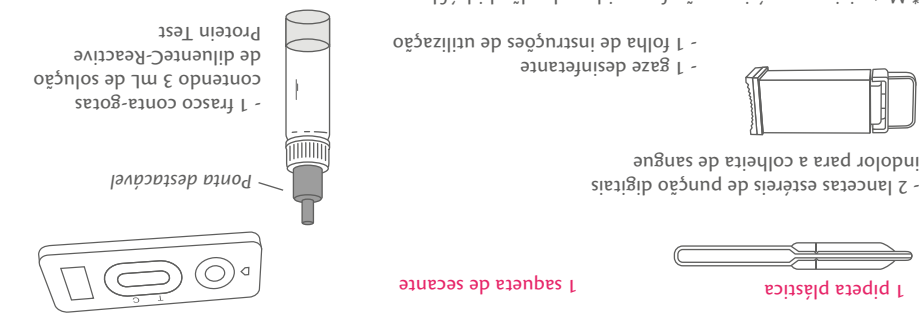
The C-Reactive Protein (CRP) is a non specific marker which is mainly produced by the liver and used to diagnose bacterial infectious disease and inflammatory disorders. The CRP is a very sensitive and fast appearing indicator which could therefore be helpful for deciding an antibiotic treatment. Depending on the CRP concentration, different lines will appear allowing the semi-quantitative interpretation of the test result.

In healthy patients, CRP concentration is lower than 8mg/L while the concentration level can go higher than 100mg/L in case of severe infection or during inflammatory process.

Intermediate levels, within 8 and 100mg/L, are concomitant with more or less mildly viral or bacterial infections that can be easily overcome by appropriate treatment ordered by your doctor.

**C-REACTIVE PROTEIN**

**PRIMA HOME TEST**

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Cut-off: <8 / 8 - 40 / 40 - 100 / >100 mg/L

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**C-REACTIVE PROTEIN TEST**  
**TEST PROTÉINA C-REACTIVA**  
**TEST PROTÉINA C-REATIVA**

Self-Test for the detection of C-Reactive Protein levels in blood

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Legal manufacturer  
 Fabricante  
 Fabricante

The C-Reactive Protein (CRP) is a non specific marker which is mainly produced by the liver and used to diagnose bacterial infectious disease and inflammatory disorders. The CRP is a very sensitive and fast appearing indicator which could therefore be helpful for deciding an antibiotic treatment. Depending on the CRP concentration, different lines will appear allowing the semi-quantitative interpretation of the test result.

In healthy patients, CRP concentration is lower than 8mg/L while the concentration level can go higher than 100mg/L in case of severe infection or during inflammatory process.

Intermediate levels, within 8 and 100mg/L, are concomitant with more or less mildly viral or bacterial infections that can be easily overcome by appropriate treatment ordered by your doctor.

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**VITREX MEDICAL A/S**