- PRECAUTIONS**
- 1) Carefully read the instructions before performing the test. The test is only interpretable if the instructions are carefully respected.
 - 2) Keep out of the reach of children.
 - 3) Do not use after the expiry date printed on the label and on the protective pouch or if the pouch is damaged.
 - 4) Follow strictly the indicated time, blood and diluent quantities.
 - 5) Store between +4 °C and +30 °C. Do not freeze.
 - 6) Do not re-use the cassette and lancet.
 - 7) External use only. DO NOT SWALLOW.
 - 8) This test is exclusively intended to in vitro diagnostic self-testing use.
 - 9) Not recommended for use by individuals taking medicines that thin the blood (anticoagulants) or by the individuals with disorders such as hemophilia.
 - 10) After use, dispose of all components in accordance with local regulations, seek advice from your pharmacist.

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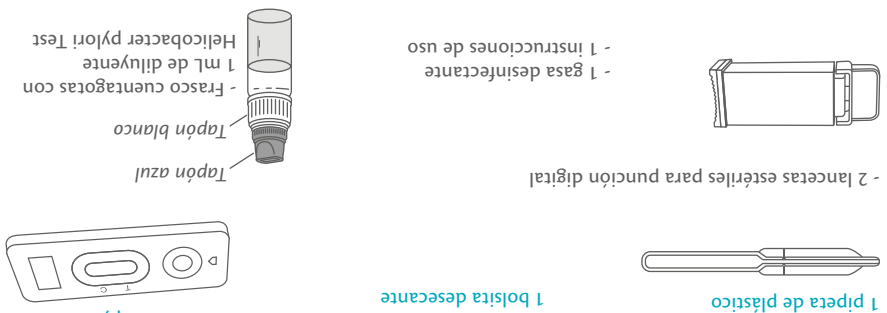
* Not provided necessary material: absorbent cotton.
 * Only open the protective pouch when you are ready to use the test.
 * The desiccant bag should not be used.


CONTENT
 THE BOX CONTAINS ALL THE NECESSARY TO PERFORM 1 H. PYLORI TEST:

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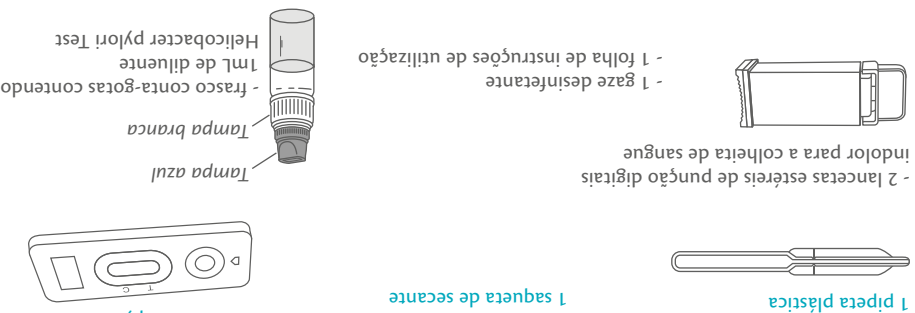The presence of the bacterium Helicobacter pylori (Hp) on the internal wall of the stomach and of the duodenum can cause chronic gastritis which can lead towards ulcers. It is proven that 90% of the people that suffers from ulcers, they have an infection caused by the Hp bacterium. This infection can increase the risk of the stomach cancer. Therefore it is very important to detect as soon as possible the infection which can be easily treated by specific antibiotics. The easiest way to detect the infection is through the presence of specific antibodies that the immune system produces after the contact with the bacterium to try to defense. The HELICOBACTER PYLORI TEST is an high specific rapid immunodiagnostic test that allows to detect the presence of the anti-Hp antibodies through a single drop of blood.

THE STOMACH AND HELICOBACTER PYLORI

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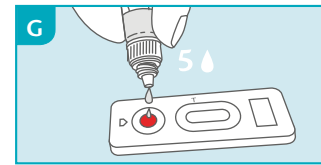
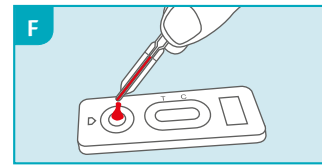
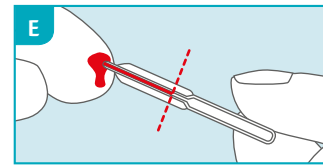
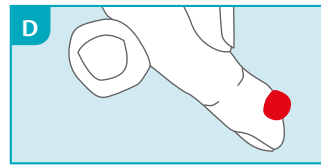
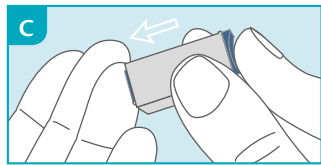
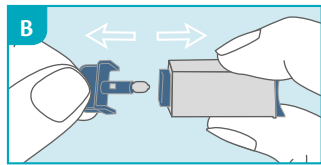
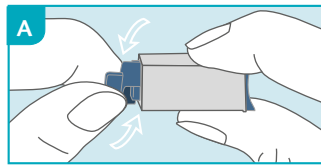
Legal manufacturer
 Fabricante
 Fabricante

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PROCEDURE

- 1) Wash your hands with soap and rinse with clear warm water.
- 2) Tear the protective pouch (from the notch) and only get out the cassette and the pipette. Dispose of the small desiccant bag.
- 3) Hold the lancet and carefully rotate 360° the protective cap without pulling it. - FIG. A
- 4) Extract and dispose the released cap. - FIG. B
- 5) Press the lancet, on the side from where the cap was extracted, against the fingertip (it is advisable the ring finger side). - FIG. C The tip retracts automatically and safely after use.
- 6) Keeping the hand down massage the end that was pricked to obtain a blood drop. - FIG. D
- 7) **Without pressing the pipette bulb**, put it in contact with the drop. The blood migrates into the pipette through the capillarity to the line indicated on the pipette. - FIG. E
You may massage again your finger to obtain more blood if the line is not reached. As far as possible, avoid of air bubbles.
- 8) Put the blood collected with the pipette into the sample well of the cassette, by pressing on the pipette bulb. - FIG. F
- 9) Wait for the blood to be totally dispensed in the well. **Unscrew the blue cap** of the plastic dropper (leave the white cap tightly screwed) and add **5 drops** of diluent into the sample well of the cassette. - FIG. G
- 10) Read the result after **10 minutes**.

RESULT INTERPRETATION

**READ THE RESULT AFTER 10 MINUTES.
DO NOT READ THE RESULT AFTER 15 MINUTES.**

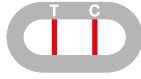
10 min.

The intensity of the line colour does not have any importance for the interpretation of the test result.


POSITIVE RESULT

Two coloured lines appear in the window under the marks T (Test) and C (Control). The intensity of the line T may be clearer than the intensity of the line C. This result means that there is the presence of the antibodies anti-Hp in blood and that you should consult a physician.


NEGATIVE RESULT

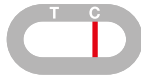Only one coloured line appears under the C mark (Control). This result means that the presence of the antibodies anti-Hp in blood was not detectable.

NON VALID RESULT

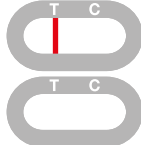No line appears or a coloured line appears under the mark T (Test) without any line under the mark C (Control). In this case, it is not possible to interpret the test, which must be considered as non valid. It is recommended to repeat the test with a new HELICOBACTER PYLORI TEST and a new blood sample.

F.A.Q. - QUESTIONS & ANSWERS

HOW DOES THE HELICOBACTER PYLORI TEST WORK?

The presence of the Hp bacterium on the gastric tissues causes the production of the specific antibodies by the immune system. There is a relation between the presence of specific circulating antibodies and the bacterium. The HELICOBACTER PYLORI TEST detects specifically the antibodies in blood and allows to presume the presence of the bacterium.

WHEN SHOULD THE TEST BE USED? The test can be performed anytime of the day. The test can be performed in case of repeated stomach and intestinal troubles (GERD, gastritis, ...)

CAN THE RESULT BE INCORRECT? The results are accurate as far as the instructions are carefully respected. Nevertheless, the result can be incorrect if HELICOBACTER PYLORI TEST gets wet before test performing or if the quantity of blood dispensed in the sample well is not sufficient, or if the number of diluent drops are less than 4 or more than 5. The plastic pipette provided in the box allows making sure the collected blood volume is correct.

HOW TO INTERPRET THE TEST IF THE COLOUR AND THE INTENSITY OF THE LINES ARE DIFFERENT? The colour and intensity of the lines have no importance for result interpretation. The lines should only be homogeneous and clearly visible. The test should be considered as positive whatever the colour intensity of the test line is.

WHAT IS THE LINE THAT APPEARS UNDER THE MARK C (CONTROL) FOR? When this line appears, it only means that the test was performing well.

IF I READ THE RESULT AFTER 15 MINUTES, WILL THE RESULT BE RELIABLE? No. The result should be read within 10 minutes after adding the diluent. The result is reliable up to 15 minutes.

WHAT DO I HAVE TO DO IF THE RESULT IS POSITIVE? If the result is positive, it means that the antibodies anti-Hp were detected in blood and that you should consult a doctor to show the test result. It might be possible that the antibodies come from a previous infection already treated. Then, the doctor will decide whether additional analysis should be performed.

WHAT DO I HAVE TO DO IF THE RESULT IS NEGATIVE? If the result is negative, it means that it was not possible to detect the antibodies anti-Hp. However, if the symptoms persist, it is recommended to consult a physician.

WHAT IS THE ACCURACY OF HELICOBACTER PYLORI TEST? The HELICOBACTER PYLORI TEST is very accurate and has been used for more than 10 years by professionals (hospitals, laboratories, ...) in the field. Evaluation reports show an overall agreement of at least 90% with reference methods.

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