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Self-Test for the detection of faecal occult blood - FOB

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Lot number
Lote de producto
Lote Produzido

Legal manufacturer
Fabricante
Fabricante

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FAECAL OCCULT BLOOD

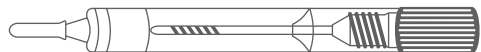
Faecal blood can indicate different gastrointestinal disorders. These disorders can be as different as ulcers, polyps, colitis, fissures and colorectal cancer. These diseases cannot be detected only by visual examination and symptoms are often silent.

Only the detection of blood in faeces can indicate a gastrointestinal disorder. Faecal occult blood in stool should be screened at least yearly starting with the age of 45 years.


BOWEL - FOB TEST is a highly specific immunological rapid test and does not require a diet period as current guaiac tests. The stool sample must be collected over 3 running days to increase the probability of intermittent bleeding detection.

CONTENT


THE BOX CONTAINS ALL THE NECESSARY TO PERFORM 1 FOB TEST:

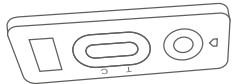
- 1 syringe containing 2 mL of extraction solution
Bowel FOB Test in a protective plastic bag

1 desiccant bag

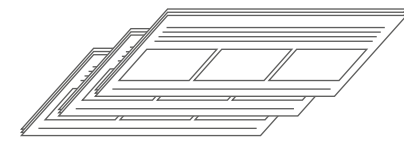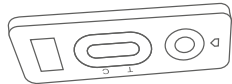
- 3 faecal sample collection paper strips

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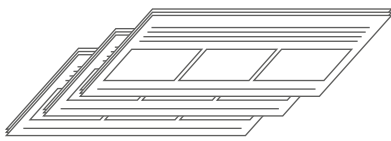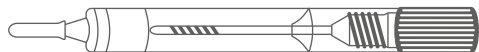
PRECAUTIONS

1) Carefully read the instructions before performing the test.

The test is only interpretable if the instructions are carefully respected.

Respect the indicated number of drops and samples.

2) Keep out of the reach of children.

3) Do not use after the expiry date printed on the label and on the protective pouch or if the pouch is damaged.

4) Store between +4°C and +30°C. Do not freeze.

5) Do not re-use the test.

6) External use only. DO NOT SWALLOW.

7) This test is exclusively intended to in vitro diagnostic self-testing use.

8) After use, all the components can be discarded in a dustbin.

Important: Women must not perform the test during menstrual periods to avoid false positive results due to menses blood.

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